CS NeoRhin Plus:

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EN NeoRhin Plus:

Read all of these instructions for use carefully because they contain important information for you.

The product, NeoRhin Plus, is a medical device. When using this device, follow the instructions stated in these instructions for use. Keep this leaflet. You may need to read it again. If you have any further questions, ask your doctor or pharmacist.

Intended Purpose: NeoRhin Plus is intended for human use to treat colds and nasal congestion. It cleans, hydrates, regenerates and treats the nasal mucosa. It creates a feeling of freshness and thus alleviates the subjective symptoms of a cold, especially the feeling of nasal congestion.

Ingredients: Hypertonic Sea salt solution (1.1 g/100 ml), panthenol, hydroxypropyl methylcellulose, eucalyptus essential oils (Eucalyptus globulus), star anise essential oils (Illicium verum), disodium edentate, sodium hydroxide.

Characteristics: Hypertonic Sea salt solution and panthenol are the main active ingredients. Hypertonic Sea salt solution helps reduce swollen mucosa based on the principle of osmosis, cleansing and hydrating them to clear the nasal passages. Panthenol regenerates the irritated and aggravated nasal mucosa, while eucalyptus and star anise oil make breathing easier by creating a pleasant feeling of freshness.

Indications: The product provides relief from colds and, nasal congestion, and cleans, hydrates, regenerates and treats the nasal mucosa. It alleviates the subjective symptoms of a cold, even of allergic origin, especially the feeling of nasal congestion.

Contraindications: The nasal spray cannot be used when there is a known hypersensitivity to any of the ingredients. It also cannot be used for illnesses of the respiratory passages linked to hypersensitivity. This includes reactions due to hypersensitivity or the intensification of lung contractions. Do not use after nasal operations or traumas in the areas surrounding the nose.

Interactions: When used in conjunction with products that eliminate swelling of the nasal membrane, use this product first. For more serious complications requiring the limited application of locally effective relief products, it is recommended that you resume use of this product after completing the application of these products by treating the area with NeoRhin spray several days afterward, if the complications do not subside.

Adverse Effects: In rare cases can lead to reactions arising from hypersensitivity and to stronger contractions of the bronchi. Any serious adverse effect that occurred in connection with the use of this medical device should be reported to the manufacturer and to the competent national authority.

Dosage and Method of Application:

The product is intended for adults and children from 8 years of age. If pregnant or nursing, consult your physician prior to use.

Dosage for adults and children from 8 years of age:

Apply the nasal spray using the applicator in each nostril several times per day, as needed. Spray once or twice in a vertical position, so that an equal amount of aerosol is discharged in every spray. The nasal spray can be used for up to 30 days as a supportive treatment for colds. You should consult your doctor if symptoms persist.

Before using, shake the bottle well and take the cover off the sprayer. Insert the end of the sprayer into the nostril and apply the product. Spray by pressing the mechanical sprayer in a vertical position, so that an equal amount of aerosol is discharged in every spray. After using, replace the lid of the sprayer. Before first use, press the sprayer several times, so that the solution is drawn into the applicator.

Warning:

- ⚠ In case of eye contact, rinse under running water.
- ⚠ One package is not to be used by more than one user, in order to avoid the spread of colds.
- ⚠ The product contains substances that are absorbed by the human body.

Storage:

Keep away from children.
Keep in the original package.

Caution: The product is not to be used after the expiration date. Once opened, do not use the nasal spray longer than 30 days.

Packaging: NeoRhin Plus is supplied in a brown glass bottle with a spray applicator. The bottle contains 30 ml of solution.

Disposal: Return any unused product to the pharmacy. Dispose of the clean packaging in a suitable recycling container.

ENEo Pharmaceuticals s. r. o., Pražská 442, 281 67 Stříbrná Skalice, Czech Republic
e-mail: info@eneo.cz

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